

Rehab/ReJoyce Software Quick Install Guide

ReHab is a software platform that is used to launch applications such as the one used by the ReJoyce device.

Prior to installing ReHab, it is recommended that the software be installed by your software/IT administrator. If you don't have an administrator, be prepared to save some password keys and have a mechanism for securely storing and retrieving them.

1. Begin the installation process by running the installation program (**rehab-setup-win64-X.XX.exe**). You will be prompted to review and accept the ReHab License Agreement.
2. Set a password for the Administrator of the software.
3. Securely save and store the Administrator password recovery Key Code for future use should the Administrator's password be forgotten.
4. It is highly recommended to leave the initial installation settings in place:
 - a. Start ReHab at Windows startup
 - b. Share anonymized usage statistics with Rehabtronics, Inc.
5. Enter the details for Registering Your Software.
6. If you select the option to **Register Now**, you will be done the Initial setup and will have the option to Run the ReHab software immediately.

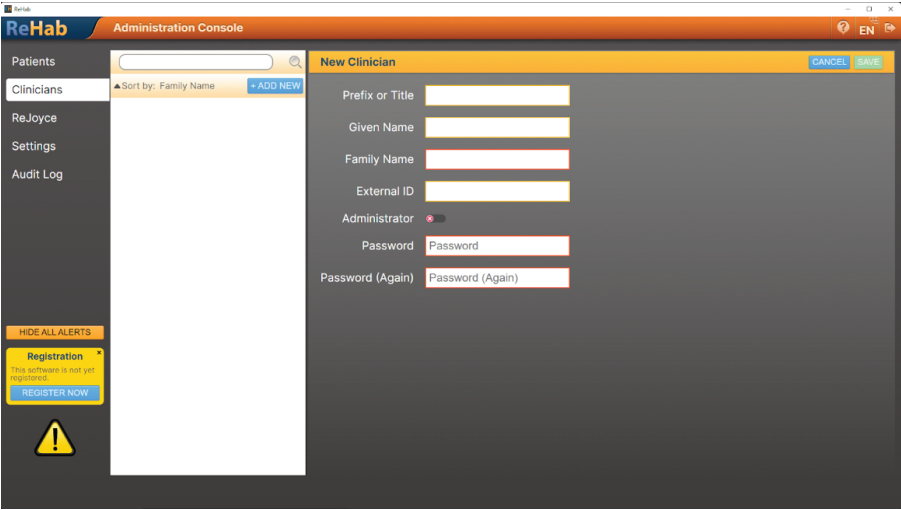


Figure 2: Screen to add new Clinician

4. Logout as the Administrator and login as one of the clinicians.
5. Ensure that the Patients tab is selected on the left pane and select the "Add New" button to create a new patient.
6. Fill in the information and if desired, the option of **Share** should be selected for other clinicians to access or view the patient information. An example of a completed patient screen is shown in Figure 3.

Overview

ReHab is a software platform that is used to launch applications such as ReJoyce. The ReHab platform software supports software use in clinical environments.

There are three user roles involved in the ReHab platform:

- Administrator (created during installation)
 - Controls software settings related to data and security.
 - Creates clinician users and can reset passwords
- Clinician
 - Can create patient profiles
 - Control Application settings (Tests and games in ReJoyce)
 - Viewing of patient data, test results and progress
 - Create new clinicians (If Assigned administrator rights)
 - Control software & application settings (If Assigned administrator rights)
- Patients
 - Takes tests for capturing results and progress
 - Engage in Games to promote recovery and improve progress.

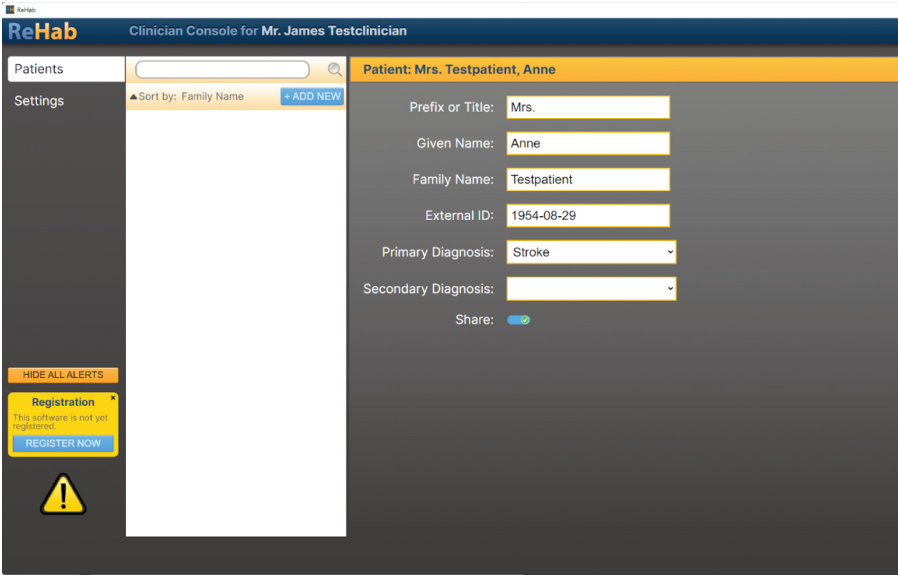


Figure 3: Example screenshot of filled in patient information

7. If not already done, ensure that the ReJoyce hardware is connected to the computer USB port.
8. After selecting one of the newly created patient profiles, the ReJoyce application can then be launched by selecting either the **Clinician Mode** or the **Patient Mode**. See Figure 4 for the Screen with the ReJoyce launch tabs.

1. If not already done, launch the ReHab software and Login as the Administrator. The screen shown in Figure 1 should be displayed after the first login:

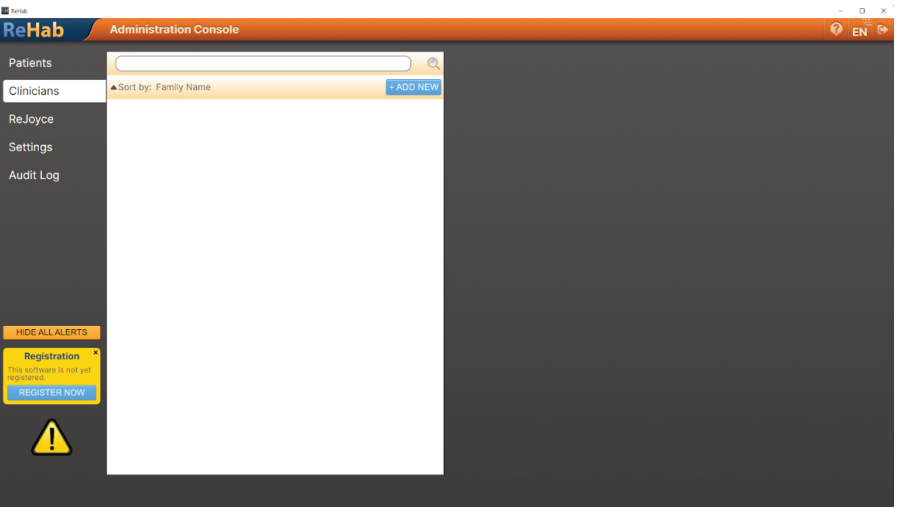


Figure 1: Initial Administrator Login Screen

2. Click on the Clinicians Tab located on the left pane.
3. Click the **Add New** button and add one or more clinicians by filling in the information. If desired, the added clinician can also be granted Administrator rights by toggling the option to on. Don't forget to click **Save** on the top right of the screen for each clinician added. See Figure 2 for the Add Clinician screen shot.

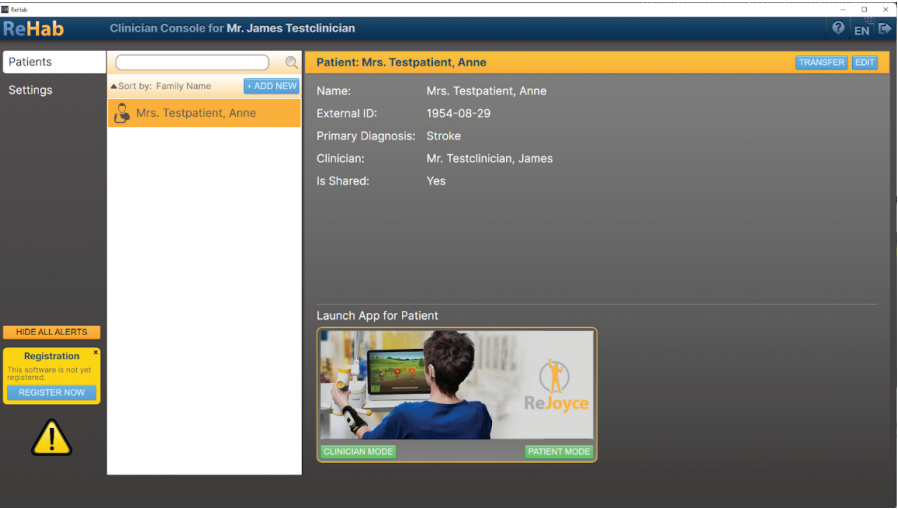


Figure 4: Screen with patient selected and ready to Launch APP

9. Whether you have a real patient or have just created a test patient to familiarize yourself with the software, select Patient Mode and opt to engage in some of the games and take one of the tests.